

# Guaranteed To Improve Suction Performance!



**1 Microbe Becomes 10,000,000 In 24 Hours  
Cleaning 24/7**

*Eliminates  
the Need to Clean  
Traps Daily*



*Use 2x a Week  
or Daily*



**Powder  
Best Value**

**Liquid  
Added  
Convenience**

**Natural &  
Non-Foaming**

**Do The  
Math!**



**BIO PURE®**  
Science That Works!

**1 x 48 oz.  
Powder**



**Leading Competitor  
10 x 67 oz.  
Liquid**

## **Bio-Pure eVac Cleaner Restore Starter Kit**

(First 2-3 weeks)

2801101 - 6 oz. Powder

2801405 - 32 oz. Liquid

**NEW!**

## **Bio-Pure eVac Cleaner Re-Order**

2801001 - 6 oz. Powder

2801002 - 14 oz. Powder

2801003 - 48 oz. Powder

2801402 - 32 oz. Liquid

**NEW!**

**Rebate!**


**FREE! \$25  
REBATE**  
voucher towards next  
qualifying purchase.  
(on Restore Starter Kits only)

**Less than  
\$9.00 USD / Month**  
Per Operatory  
On Re-Orders

### NEW USER

### Initial eVac System Bio-Pure Restore Treatments

#### Per 2 Operatories

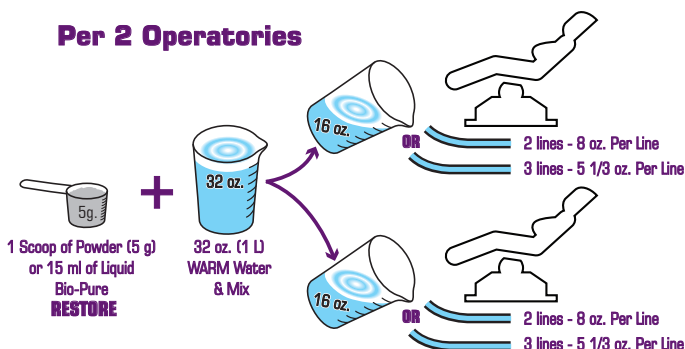

#### RECOMMENDED PRODUCT:

**2801101 Bio-Pure POWDER Starter Kit** (order 1 per 4 Ops)

**2801405 Bio-Pure LIQUID Starter Kit** (order 1 per 4 Ops)

Each Starter Kit Includes: Restore Bio-Pure, FREE HVE-SE Dispenser, \$25 rebate coupon towards next Bio-Pure purchase (powder: 3 x 14 oz. or 1 x 48 oz., or liquid: 4 x 32 oz. or 1 x 1 gal).

#### INSTRUCTIONS; Per 2 Operatories:

- Begin the restore protocol at close on the last day of the work week.
- Add 1 scoop of powder (5 g) or 1/2 oz. (15 ml) of liquid Bio-Pure to 32 oz. (1 L) of WARM WATER, & mix.
- If operatory has 2 lines, aspirate/vacuum 8 oz. (0.25 L) per line.
- If operatory has 3 lines, aspirate/vacuum 5 1/3 oz. (0.33 L) per line.
- Inspect traps twice a day and clean as necessary.

#### DURATION OF TREATMENT BASED ON AGE OF PRACTICE;

1 to 4 Years - Repeat daily for 5 business days.

4 + Years - Repeat daily for 10 business days.


#### Slow Suction or 15+ Years;

- Restore **ONLY** two operatories at a time.
- Begin protocol with the two operatories nearest the pump.
- Follow the above instructions daily for 10 business days or once Biofilm/plaque release ends at the chair side traps.
- After first 2 operatories are clear, proceed with the next 2 ops working away from pump after 10 business days.
- For optimal results, involve a technician during initial Restore introduction (eg. visit following Monday & drop off back-up amalgam filter).

### EXISTING USER

### Ongoing eVac System Maintenance Treatments

#### Twice Weekly Maintenance Protocol Per 1 Operatory

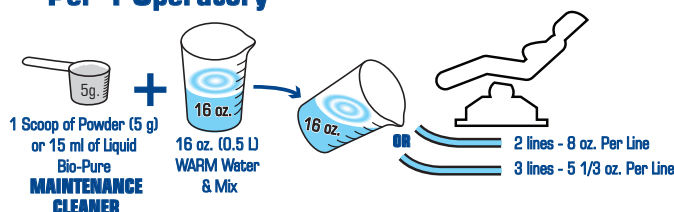

#### RECOMMENDED PRODUCTS:

**2801001 - 6 oz. POWDER Maintenance Cleaner** (4 Ops, 1 mo.)

**2801002 - 14 oz. POWDER Maintenance Cleaner** (4 Ops, 3 mos.)

**2801003 - 48 oz. POWDER Maintenance Cleaner** (4 Ops, 9 mos.)

**2801402 - 32 oz. LIQUID Maintenance Cleaner** (4 Ops, 2 mos.)

#### Twice Weekly Maintenance Instructions; (per 1 operatory)

- At closing add 1 scoop of powder (5 g) or 1/2 oz. (15 ml) of liquid to 16 oz. (0.5 L) of WARM WATER & mix.
- Aspirate/vacuum into operatory lines dividing 16 oz. solution evenly between each line (8 oz. for 2 lines, 5 1/3 oz. for 3 lines). Proceed one line at a time.
- Procedure requires 2 treatments per week (end of day).

#### -OR-

#### Daily Maintenance Instructions; (per 2 operatories)

- At closing add 1 scoop of powder (5 g) or 1/2 oz. (15 ml) of liquid to 32 oz. (1 L) of WARM WATER & mix.
- Aspirate/vacuum into 1 operatory lines dividing 16 oz. solution evenly between each line (8 oz. for 2 lines, 5 1/3 oz. for 3 lines). Proceed one line at a time.
- Treat each operatory daily at the end of the work day.

**Additional Best Practices:** Any extra Restore product may be used for the Maintenance Protocol. You may continue your regular chemical cleaning protocol for operatories not using Bio-Pure. Never place a wet scoop into Bio-Pure container. Re-seal lid to ensure freshness. Store out of direct sunlight and not in refrigerator. Rinse dispenser and pour residual mixture down sink drain to put microbes to work, and then store dispenser with the lid off. After system restore, pump settings may need to be re-set to original OEM specifications. Also, monthly mix 1 scoop of Bio-Pure with 24 oz. (0.7 L) WARM WATER and pour down cuspidor or other drains to keep clean.

#### Daily Maintenance Protocol Per 2 Operatories

