


Lime-Lite™ Light Cure Cavity Liner

Reduces Pulpal Sensitivity

Contains Hydroxyapatite
Universal Shade • Radiopaque
Fluoride Releasing

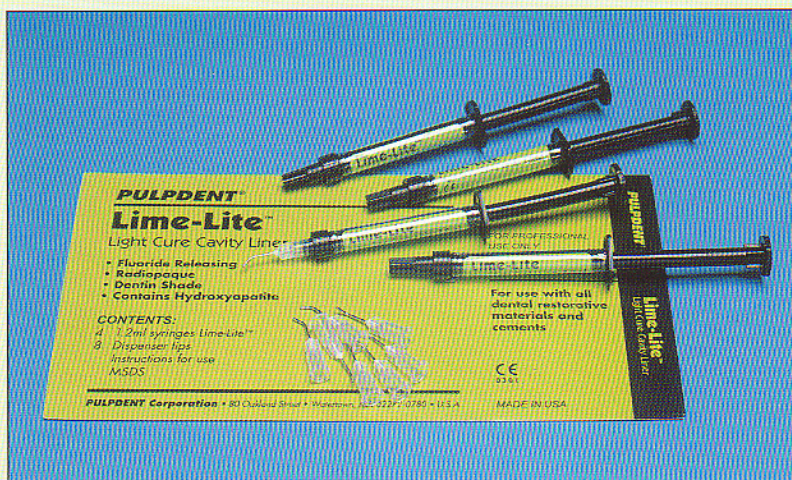

**Specially formulated for use
with today's adhesive dentistry**

Lime-Lite™ has all the following benefits:


- Light cure base/liner – sets extremely hard
- Chemically bonds to adhesives and composites
- Convenient syringe dispensing
- Contains hydroxyapatite
- Releases favorable calcium, hydroxyl, phosphate and fluoride ions
- Stimulates secondary dentin formation
- Fluoride releasing
- Radiopaque
- Insoluble in water and oral fluids

LIME Kit: 4 x 1.2 mL syringes + 8 dispenser tips

LIME-3 3mL syringe only

LTN Dispenser Tips, pink, 20 gauge, pkg/20

LTL Dispenser Tips, pink, 20 gauge, pkg/100

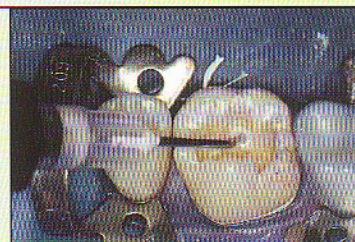


Place Lime-Lite in the cavity prep.




Light cure Lime-Lite 20-30 seconds.



Etch the cavity prep with Etch- Rite
38% phosphoric acid.

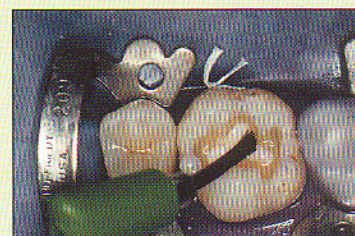

Apply DenTASTIC UNO to moist dentin
surface for light cure, or
UNO + DUO for dual cure procedures.

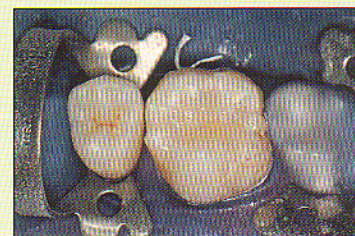

Final restoration – incrementally
layered with packable composite.

PULPDENT Corporation

80 Oakland Street • Watertown, MA 02471-0780 • U.S.A. • (617) 926-6666 or (800) 343-4342 • Fax (617) 926-6262
Visit Pulpdent Online at <http://www.pulpdent.com> • Email: pulpdent@pulpdent.com