

DisCide® ULTRA Shelf Life

Frequently Asked Questions

Q1: What is the shelf life for DisCide Ultra?

For unopened products, the shelf life is 2 years for towelettes and 3 years for quart spray bottle and gallon container from the manufacturing date.

Q2: Where can I find the manufacturing date code to DisCide Ultra surface disinfectant products?

The manufacturing date is printed at the bottom rim surface of the towelettes canister, quart spray bottle and gallon container.



Q3: How do I interpret the date code printed on DisCide Ultra towelette canisters, quart bottles and gallon containers?

The manufacturing date code format is presented in the chart below.

Product (Ref#)	Date Code Format	Date Code / Expiration Example	Date Code Interpretation and Representation
Towelettes Canister (10DIS & 60DIS)	JJYB TT:TT (New Mfg. Date format effective April 2022)	08424 18:34 Day of Year = 84* Year = 2022 Batch Number = 4 Time = 18:34 / 6:34 PM	JJJ = Julian Date of Manufacture* Y = Last Digit of Year of Manufacture B = Batch Number TT:TT = Military Time
		<u>or</u>	LOT = Static Text
	LOT YYDDD WWWW TTTT	LOT 18270 12200 1315 Year = 2018 Day of Year = 270* Work Center = 12200 Time = 13:15 / 1:15 PM Mfg. Date = 9/27/2018 Expiration Date = 9/27/2020	YY = Last two digits of year DDD = Julian Date* WWWWW = 5-digit work center TTTT = 24-hour time
			*Julian calendar of the year chart link: http://www.scp.byu.edu/docs/doychart.html
Quart & Gallon Bottles (3565Q & 3565G)	YYMMDD-XXXXXXXXX (New Mfg. Date format effective December 2018)	181218-00000001 Year = 2018 Month = 12 (December) Day = 18 Batch = 1 Mfg. Date = 12/18/2018 Expiration Date = 12/18/2021	YY = Two digits of the year MM = Two digits of the month DD = Day of the month XXXXXXXXX = Batch
			